DropSafe™

Safety Pen Needle

Sterile, single-use safety pen needles intended for use with pen injector devices for the injection of drugs.

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Read the needle gauge (G) and length (mm) on the seal to make sure you are using the right safety pen needle for you or ask for assistance if you have impaired vision. Also, do so if you are a healthcare professional, a caretaker, or family member assisting a patient.

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Recommended injection technique:

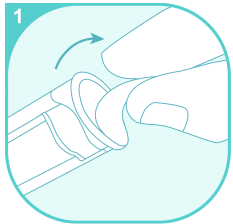
Make an injection into flat skin vertically at 90° angle. Alternative injection technique: you may also make a skin fold if recommended by your healthcare professional.

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
1 Check the seal. If it is missing or damaged, use a new safety pen needle. Remove the seal. Check the needle, use another one if it appears to be damaged.

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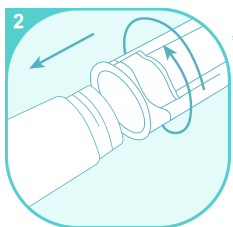
2 PUSH the safety pen needle straight onto the pen holding it in one axis with the pen and twist until it is tight. DO NOT over-tighten.

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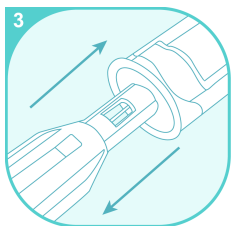
3 Remove the outer cap. Make sure not to press the shield before use as any pressure on the shield may cause the needle to lock and the safety pen needle to be unusable. USE a new safety pen needle if the red stripe is visible in the viewing window or the needle appears to be damaged. Keep the outer cap.

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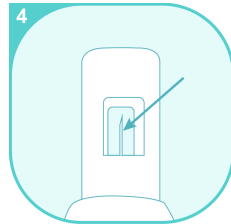
Conservez le capuchon externe.

4 Check if the needle is visible in the viewing window. Use a new safety pen needle if the needle is not visible in the viewing window.

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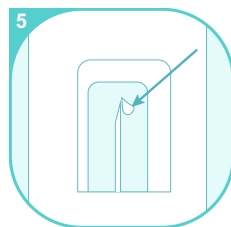
5 Perform a priming test if recommended by the pen injector device manufacturer. A drop of liquid should appear on the needle tip visible through the viewing window. Use a new safety pen needle if the priming test was unsuccessful.

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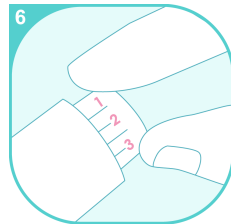
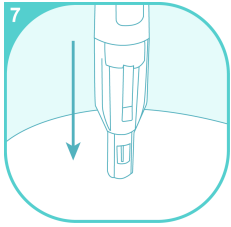
Instructions for use / Instrucciones de uso / Mode d'emploi

Insert the needle into the flat skin at a 90° angle in one continuous movement until the shield is fully recessed. Deliver the dose. DO NOT remove the needle until the dose has been completely delivered. Refer to the instructions for use of the pen injector device manufacturer for specific hold time.

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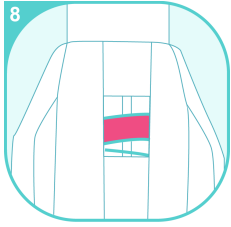
When dose delivery is complete, remove the needle from the skin. The shield will automatically cover the needle and lock it. A red stripe will appear in the viewing window. The needle is locked and cannot be reused. **Note:** Liquid may fall back into the needle shield during priming, which may result in fluid being left on the skin after injecting.

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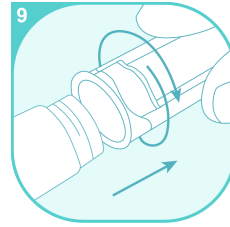
Twist off the needle to remove it from the pen injector device. It is recommended that you cover the back end with the outer cap.

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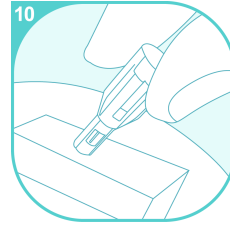Discard used safety pen needle into a sharps container.

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Caution / Precaución / Avertissements:

Keep out of reach of children.

Avoid infections: Before making an injection, clean and disinfect the injection site. Always follow your facility sharps injury prevention guideline or local regulations. Do not use if the seal or outer cap is damaged or missing, or if the shield or needle appears to be damaged.

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Refer to the instructions for use of the drug and pen injector device for specific storage and handling instructions.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent/government agencies, as applicable.

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Manufactured for:

HTL-STREFA S.A.
ul. Adamówek 7, 95-035 Ozorków, Poland
T: +48 42 270 00 10, F: +48 42 270 00 20
www.htl-strefa.com, info@htl-strefa.pl
Made in Poland

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© 2022 HTL-STREFA, Inc., a subsidiary of HTL-STREFA S.A., a company of MTD Group. All rights reserved. Unless otherwise noted the DropSafe logo and all other trademarks are the property of HTL-STREFA S.A. and/or its affiliates. A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to prescribe, recommend, or use a particular product when treating a particular patient.

820/8161/04/2022
Page/Página/Page 2 of 2