


# DropSafe<sup>™</sup> Sicura<sup>™</sup>

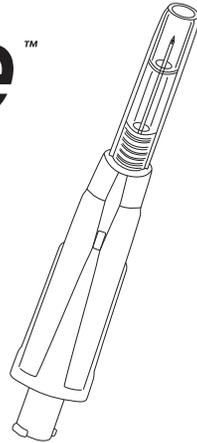


Sterile, single-use passive safety injection needle to be used in combination with a (pre-filled syringe for subcutaneous and intramuscular injection.

## CAUTION:

- Cannot be used for drawing up drugs from a vial / ampule. Drugs drawing up from vial / ampule could damage the cannula and its tip.
- Always verify the device's integrity before use.
- Check the expiration date. If device is past expiration date, use a new device.
- Sterile contents. Check the blister pack, and the needle itself. If damaged, use a new device.
- Dispose into an approved sharps container in accordance with applicable regulations and facility guidelines.
- Store in a dry place away from chemical substances.
- If any part of the illustrated components are missing, use a new device.
- Do not touch the safety shield at any time to avoid premature activation of the sharps injury prevention feature.
- Consult the drug monograph prior to selecting appropriate device size.

## PREPARING FOR AN INJECTION WITH DROPSAFE<sup>™</sup> SICURA<sup>™</sup>

1. Wash your hands before using the device.
2. Select appropriate personal protective equipment in accordance with facility protocol.
3. Fill the syringe with the drug using a blunt fill needle - do not use DropSafe Sicura for filling the syringe or use a pre-filled syringe.  
**NOTE: Do not use DropSafe Sicura for drawing up drugs from vial / ampule. Drawing up of drug from vials / ampules could damage the cannula and its tip irreparably!**
4. Review the professional drug monograph to select the appropriate DropSafe Sicura needle size, and identify proper injection technique for the patient.
5. Check the integrity of the blister and device before the use.

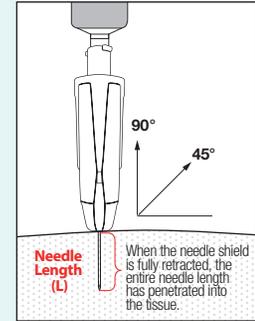


### Note!

Please review the illustration on the left when selecting the appropriate needle gauge & length listed in the table.


This illustration is solely intended to provide visual representation of the needle length.

## Instructions for use


Read the instructions for use and use the device accordingly.  
Review the drug's monograph and follow your facility's recommended protocol for drug administration.

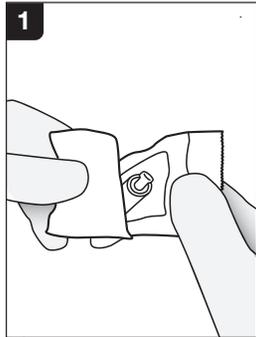


Verify device integrity before use. Disinfect the administration site using an alcohol prep pad.  
Use aseptic technique. Peel blister of DropSafe Sicura halfway.

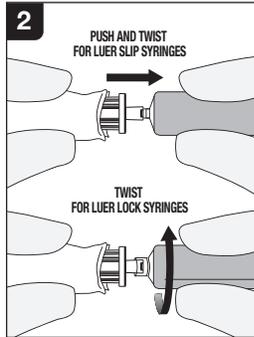


Secure DropSafe Sicura to a syringe filled with the drug.  
**Note!** Leave the outer cover on until ready to administer.

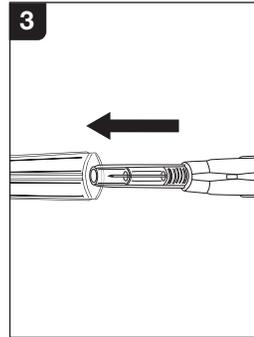


Pull the outer cover straight off the needle. Remove the air or prime the needle, if required.  
The device is now ready for use.  
In the event DropSafe Sicura becomes detached from the syringe prior to injection, verify the syringe contents before administration.  
**Note!** Do not touch the needle transparent shield.

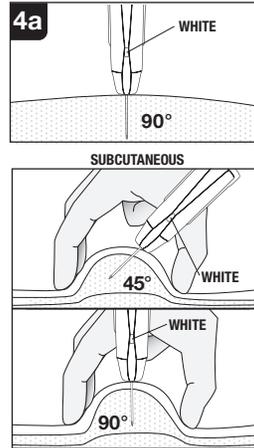

Select injection site and position DropSafe Sicura at a 45° or 90° angle at the injection site to administer the drug.  
Use with/without a skin fold (pinch-up) technique according to the professional drug monograph's recommendation.  
**Note!** Before use, the needle use indicator is white.

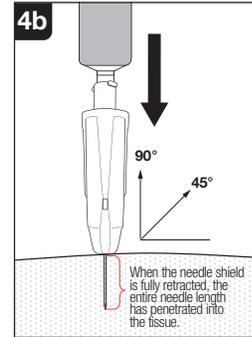

Hold the desired angle of 45° or 90° while pushing the needle in one continuous movement to administer the drug. Continue to maintain the injection angle when withdrawing the needle after drug delivery is complete.  
The safety mechanism will activate automatically during needle withdrawal from site of administration.  
**Note!** During injection, the shield will retract into the device. Hold the device in position during injection to prevent push-back from the shield and to ensure full drug delivery. No audible click or palpable difference will be observed during safety mechanism activation.

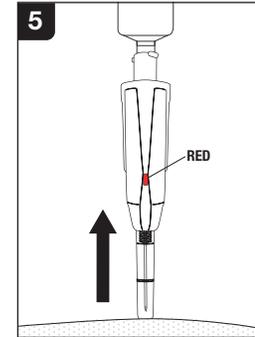

After the full dose is administered, remove the needle from the injection site.  
**Note!** After injection, the needle use indicator turns red. The needle automatically locks in the safety shield. No additional safety activation step is required.



Dispose of DropSafe Sicura and syringe into an approved sharps container in accordance with applicable regulations and facility guidelines.

Scan here to watch Instructions for Use prior to use

Manufactured by:  
PIKDARE S.p.A  
Via Saldarini Catelli 10,  
22070 Casnate con Bernate (CO), Italy  
T: +39 0317297832  
Made in Italy

Distributed by:  
HTL-STREFA, Inc.  
Woodstock, GA USA  
T: +1-770-528-0410, F: +1-770-528-0411  
www.dropsafe.info, info@htl-strefa.com

© 2022 HTL-STREFA, Inc., a subsidiary of HTL-STREFA S.A., a company of MTD Group. All rights reserved. Unless otherwise noted the DropSafe logo and all other trademarks are the property of HTL-STREFA S.A. and/or its affiliates. A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to prescribe, recommend, or use a particular product when treating a particular patient.


Customer service:  
1-877-660-1900  
info@htl-strefa.com  
940/12/2022

MFR#	Size	QTY:
02094000300130*	30G x 1/2" (0.3 mm x 13 mm)	50 pcs / box 4 boxes / case
02094000270130*	27G x 1/2" (0.4 mm x 13 mm)	
02094000250160*	25G x 5/8" (0.5 mm x 16 mm)	
02094000250250	25G x 1" (0.5 mm x 25 mm)	
02094000230250*	23G x 1" (0.6 mm x 25 mm)	
02094000220250*	22G x 1" (0.7 mm x 25 mm)	
02094000210250*	21G x 1" (0.8 mm x 25 mm)	
02094000180250*	18G x 1" (1.3 mm x 25 mm)	

\*Available in 2024



# DropSafe<sup>™</sup> Sicura<sup>™</sup>

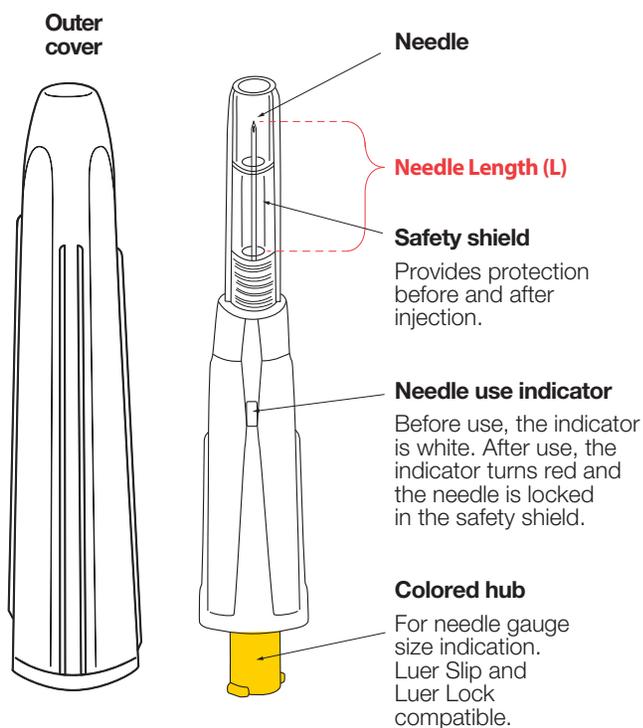

DropSafe Sicura passive safety injection needles are available in eight different needle gauges (diameters) and lengths.

Recommendations for selecting gauge and needle length:

Needle hub color	Size Gauge (G) × Needle Length (L - inches) Diameter (mm) × Needle Length (L-mm)	Suggested for SC injection*	Suggested for IM injection*
Yellow	<b>30G × ½"</b> (0.3 mm × 13 mm) Use with 3mL syringe maximum	45° or 90° with pinch-up	
Grey	<b>27G × ½"</b> (0.4 mm × 13 mm) Use with 3mL syringe maximum	45° or 90° with pinch-up	
Orange	<b>25G × ⅝"</b> (0.5 mm × 16 mm)	45° or 90° with pinch-up	
Light Orange	<b>25G × 1"</b> (0.5 mm × 25 mm)		90° without pinch-up
Blue	<b>23G × 1"</b> (0.6 mm × 25 mm)		90° without pinch-up
Black	<b>22G × 1"</b> (0.7 mm × 25 mm)		90° without pinch-up
Green	<b>21G × 1"</b> (0.8 mm × 25 mm)		90° without pinch-up
Pink	<b>18G × 1"</b> (1.3 mm × 25 mm)		90° without pinch-up

\* These are suggested indications. The healthcare professional must use their professional judgment to determine which DropSafe Sicura version to use for a particular drug and/or patient.

The pinch-up technique is the common name for the injections performed with the skin fold.

Scan here to watch  
Instructions for Use  
prior to use

Manufactured by:  
PIKDARE S.p.A  
Via Saldarini Catelli 10,  
22070 Casnate con Bernate (CO), Italy  
T: +39 0317297832  
Made in Italy

Distributed by:  
HTL-STREFA, Inc.  
Woodstock, GA USA  
T: +1-770-528-0410, F: +1-770-528-0411  
www.dropsafe.info, info@htl-strefa.com

© 2022 HTL-STREFA, Inc., a subsidiary of HTL-STREFA S.A., a company of MTD Group. All rights reserved. Unless otherwise noted the DropSafe logo and all other trademarks are the property of HTL-STREFA S.A. and/or its affiliates. A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to prescribe, recommend, or use a particular product when treating a particular patient.

Customer service:  
1-877-660-1900  
info@htl-strefa.com  
940/12/2022

MFR#	Size	QTY:
02094000300130*	30G × ½" (0.3 mm × 13 mm)	
02094000270130*	27G × ½" (0.4 mm × 13 mm)	
02094000250160*	25G × ⅝" (0.5 mm × 16 mm)	
02094000250250	25G × 1" (0.5 mm × 25 mm)	50 pcs / box
02094000230250*	23G × 1" (0.6 mm × 25 mm)	4 boxes / case
02094000220250*	22G × 1" (0.7 mm × 25 mm)	
02094000210250*	21G × 1" (0.8 mm × 25 mm)	
02094000180250*	18G × 1" (1.3 mm × 25 mm)	

\*Available in 2024

