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Pen needles are sterile, single-use disposable devices. The pen needle must be changed and properly disposed of after each injection.

TO REDUCE PAIN USE NEW, SHORT, THINNER NEEDLES AND ALWAYS FOLLOW YOUR PHYSICIAN'S INSTRUCTIONS.

Use a new pen needle at each injection to prevent:

- Inaccurate dosage. If the needle is left on the pen between injections, the composition of the medication may be altered.
- Air getting into the cartridge through the needle. The air bubbles reduce the normal flow of medication during the injection, with the consequent risk of a reduced dose of the drug.
- The crystallization of the medication. If the needle is left mounted on the pen, the medication remaining inside the pen can transform into crystals, and partially or completely obstruct the needle, causing difficulty in or inability to carry out the injection.
- Contracting infections. Once used, the pen needle is no longer sterile.
- Experiencing a more painful injection. During the injection, the lubricant, put on the needle is completely removed and the tip of the needle undergoes a progressive deformation, losing its original shape. Often the tip of the needle becomes a "hook" shape, causing increased pain and possible tearing of the skin.

Avoid infection. Do not re-use or share pen needles. Do not use pen needles if the individual packaging is damaged, the seal is missing, or the needle appears to be damaged. To avoid accidental pricking with needles, never handle someone else's pen needle.

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WARNING
ADVERTENCIA
AVERTISSEMENT

Remove the inner protective cap by pulling it straight out to prevent bending of the needle and/or damaging the needle tip.

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Novo Nordisk
FlexPen, Novopen 3, Novopen 4, Novopen Echo, Novopen Junior, Victoza Pen, Innolet, Novopen 5, FlexTouch, Levemir® FlexPen®, NovoLog®, FlexPen®, NovoLog® Mix 70/30 FlexPen®, Tresiba® FlexTouch® U-100, Tresiba® FlexTouch® U-200

Oven Mumford
Autopen Classic, Autopen 24

Ypsomed
Ypsopen

Berlin Chemie
Berlpen Aro2

Widok wewnętrzny ulotki / Internal view of the leaflet

Here are the steps to follow to ensure accurate use of the pen needle:

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Check the expiration date of the pen needle on the paper tab. Do not use if expired.

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Push the pen needle straight onto the pen injector device and twist until it is tight. Do not push the pen needle at an angle.

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Remove the outer protective cap and set it aside. Do not discard. You will need it later to remove the pen needle from the pen injector device.

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Remove both the outer cover and the inner needle cover before an injection (See Fig. 5 and Fig. 6, respectively). If both the outer cover and the inner needle cover are not removed before use, the medication or dose will not be injected, which may result in serious injury or death.

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Prime your pen injector device per your pen injector device's instructions for use to prevent incorrect dosage caused by air in the cartridge or a clogged needle. If no drug flow is seen after several attempts, attach a new pen needle.

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Set your drug dose.

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When using Droplet Micron, inject into the skin vertically (at a 90° angle) straight and without a skinfold. Inject slowly. Do not withdraw the needle immediately after injection. Do not change the direction of the pen needle while it remains in the body as this can result in bending or breaking of the needle.

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WARNING
ADVERTENCIA
AVERTISSEMENT

Place the larger outer protective cap over the pen needle.

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Remove the pen needle from the pen injector device after each injection to prevent air from entering the cartridge and drug from leaking out and dispose of safely according to local regulations.

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